



Single Page Reference Sheet

Dataset Interaction


is active	3D window interaction	Dataset interaction	2D window interaction
Dolly (pull dataset closer)	left click + right click Ctrl + left click	Reset view	Right click and select Reset view
Rotate	Left click	Flip 180 degrees horizontal	Right click and select Mirror image - Horizontally
Spin	Alt + left click Shift + right click	Flip 180 degrees vertical	Right click and select Mirror image - Vertically
Translate	Shift + middle click	Change slice	Slider
	Middle click	Zoom	Right click Ctrl + right click
Alt + middle click	Alt + right click		
Zoom	Right click	Width	Left click and horizontal movement
	Ctrl + right click Alt + right click		Level

is active	3D window interaction
Width	Left click and horizontal movement
Level	Left click and vertical movement

Widget Interaction



Widget interaction	Action
Left drag	Move widget along x,y axis
Ctrl + left drag	Move widget along x,z axis

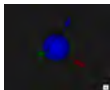

Position controls (3 colored arrows + arcs)	Arrow Action	Arcs Action
Left drag	Move along arrow axis	Rotate widget & control widget
Shift + left drag	Move along arrow axis	Rotate control widget



Rotation controls (3 colored circles)	Arcs Action
Left drag	Rotate widget & control widget
Shift + left drag	Rotate control widget



Single Axis controls (1 colored arrow)	Cone Action	Line Action
Left drag	Rotate widget & control widget	Drag along axis
Shift + left drag	Rotate control widget	Drag along axis

Near Plane functionality	Add Near Plane checkbox in the Cropping Panel. Window - Add Near Plane. Right Click in the 3D view and select Add Near Plane. Sliders in the Cropping panel
Orthogonal Planes	2D window - drag plane Center plane - right-click slider and select Center slider
Oblique Planes	3D window - single axis control widget 2D window - dragging plane or endpoints
Placing a point	"Picking Mode" and double clicking the location. The point will always be placed on the plane closest to the camera.
Placing measurements	Select measurement type and double click the location To finish placing a measurement, double click the last placed point or middle mouse button.
Maximize view	CRTL-M
Full screen view	CRTL-F